EN

Surgi-Cuf® REUSABLE BLADDERLESS BLOOD PRESSURE CUFFS

INDICATIONS FOR USE

Bladderless Blood Pressure Cuffs for use with non-invasive manual or electro-mechanical sphygmomanometers.

DESCRIPTION

Ethox Surgi-Cuf® blood pressure cuffs are manufactured in conformance to AAMI specification SP10 and EN 1060, and color-coded for size selection.

7000 Series Reusable Bladderless Blood Pressure Cuff Size Selection

Size	Color	Range
NEONATAL	Orange	8 – 14 cm
CHILD	Green	13 – 20 cm
SM. ADULT	Light Blue	18 – 26 cm
ADULT	Dark Blue	26 – 35 cm
ADULT LONG	Dark Blue	32 – 42 cm
LG. ADULT	Burgundy	32 – 42 cm
THIGH	Brown	42 – 50 cm

INSTRUCTIONS FOR USE

- Blood pressure measurements should be taken by trained personnel following established procedures.
- Select a cuff with a range spanning the patient's limb circumference.
- Remove clothing from limb before positioning the cuff. Wrap the cuff around the patient's limb, aligning the artery mark over the brachial or femoral artery on single tube cuffs and between artery marks on two tube cuffs. Close the hook and loop fasteners. Make sure the INDEX LINE of the cuff lays between the cuffs MIN RANGE MAX marking. Select a different size cuff when the index line falls outside the range.
- Attach cuff connector to mating connector on sphygmomanometer.
- Take blood pressure. For electronic monitoring, be sure to follow the monitor manufacturer's instructions.

CAUTION

- When using luer lock connectors, be careful not to connect cuffs to intravascular fluid systems.
- Do not inflate the cuff when not on the patient.
- Use of the wrong sized cuff can effect the accuracy of the measurement

CLEANING AND INSPECTION

- Prior to use, check that the cuff, tubing, and hose are clean and free of damage. Replace cuff when aging, tearing, or weak closure is apparent.
- Wipe or brush off any visible soil.
- Apply a mild detergent to a damp cloth and wipe clean.
- Inspect the tubing and bladder for signs of degradation, cracks, holes, or damage after each cleaning.
- Do not allow fluid to enter the tube(s). This may affect pressure accuracy or damage equipment.

DISINFECTANT

- After cleaning, spray cuff with a 1:100 dilution of hypochlorite solution (500 – 600 ppm free chlorine) until surface is saturated.
- Let stand for 2 minutes
- Wipe dry with a clean cloth
- Disinfectant may cause discoloration.

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